


Troubleshooting: Is my DDS Brace leaking air?

Please follow these simple steps to make sure your DDS product is working efficiently. Read the User Manual thoroughly for proper use.

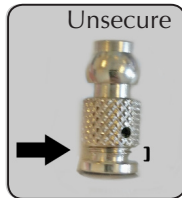



Air Valve

Turn the air valve clockwise until the valve is fully locked.



Extension Hose & Air Valve

The Extension Hose is an added accessory for easier access to the air valve. Firmly secure both the Hose and Valve.

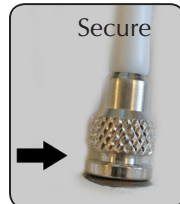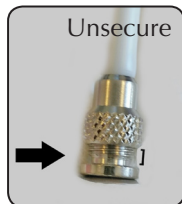



Extension Hose to Brace

Firmly secure the Extension Hose to your DDS Brace.



Gauge on the Hand Pump

The Gauge can unfasten depending on how it is used. Please review the diagram to be sure it is fully secured at all times.

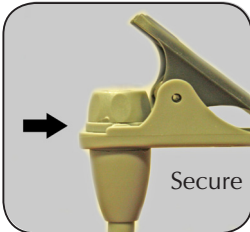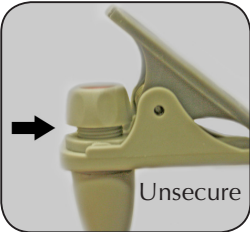

Connection Screw on the Hand Pump

The Connection Screw also must be fastened at all times for proper inflation.



Black O Ring inside the Air Valve

The Black O Ring inside your Air Valve is essential for your Brace and Hand Pump to work properly. If you lose the Air Valve or Black O Ring, please contact 888-495-7440 for a free replacement.